

Omni v.2 / Omni v.2 Pro Cylinder Conversion Kit Instructions

p/n M4001X



M4001X Cylinder Conversion Kit Components



Cylinder Pillow

Cylinder Divider

The following instructions will assist you in converting your Omni v.2 or Omni v.2 Pro to secure an MD or MJD size medical oxygen cylinder.



Omni v.2/ Omni v.2 Pro

1. Open the main pocket of your Omni and remove all the internal dividers and sliders.



2. Insert the Cylinder Divider so that it lines up with the cylinder access flap on the left side of the main pocket.



3. Insert the small slider dividers on the lower portion of the Cylinder Divider. You can customize the size of the pockets for your team requirements.



4. Insert an MD or MJD size medical oxygen cylinder into the upper portion of the Cylinder Divider.



5. Insert and wrap the Cylinder Pillow across the cylinder. 3 strips of loop (fuzzy) velcro should appear on the top of the cylinder if the pillow is installed correctly.



6. Congratulations, your installation is now complete. You now also have the option of adding stickit strips to the 3 strips of loop (fuzzy) velcro.

